HPTN 035 Moshi Tanzania

Quality Assurance for Rapid Tests

RAPID TESTS

- Performed at satellite laboratory at Mawenzi Clinic
 - Reduces turnaround time
 - Simplifies logistics
 - Preserves specimen integrity

RAPID TESTS

- Urine Pregnancy Testing
 - Quidel QuickVue One Step hCG Urine Test
- Urine Macroscopic Examination
 - Quidel QuickVue UrinChek (Nitrite and Leucocytes)
- Wet Mount for Bacterial Vaginosis

RAPID TESTS

- HIV Testing
 - Abbott Laboratories Determine HIV-1/2
 - Orasure Technologies OraQuick Rapid HIV-1 Antibody Test

QUALITY ASSURANCE

Quality Control

Training and Competency Testing

Ongoing Monitoring

- Urine Pregnancy
 - Quidel Positive and Negative Control
 - Once per Box (Every 25 tests)
 - Documentation on Lot log sheet and Urine Results Register

- Urine Macroscopic
 - QC only recorded for Nitrites and Leucocytes
 - Sentry Normal and Abnormal
 - Monthly
 - Documentation in the Urine Results Register and Lot Log Sheets

- Wet Mount for Bacterial Vaginosis
 - Commercial QC materials not available
 - Rely on competency of technologists
 - May implement random review of slides

- OraQuick HIV
 - Orasure Positive and Negative controls
 - Every 100 kits
 - Documentation in Blood results register and Lot Log Sheets

- Determine HIV
 - Known Positive and Negative frozen sera
 - Every 100 kits
 - Documentation in Blood results register and Lot Log Sheets

IIPTN LABORATORIES REACENT LOT USE/OC LOG Abbot Determine® BIV-1/2 Test Determine Negative Condrol Posttive Control Communic Stripel Los Testing Institute Chase Huffer Started Date Samer Loo'Acce Rapid Louis Accept Lut A /Exp date QC must be performed for each new 'of of Determine strips, QC not required for Chase Buffer. Controls may be ren more frequently if problems are suspected or unusual results are encountered.

Training and Competency

	SCHONTRIALS NET		
TEST PACTIFICATION DE L'ANTI-L	CTeaning HV-12	.Na	
TAUTH COPIE	ZENCA DIP TING		
Sponter Manification e	Engloyer Hamal	Knowa Vála	Namelia Acceptatio
-			Yes N
10000			Yzz No
			Ya No
			Ea_he
			YesX2
		-	X82 X7
====	altaren 1960 (Oga		
	neconnected:		
discount yes		Date	

Ongoing Monitoring

COMPREHENSIVE QUALITY ASSUI	RANCE PROG	RAM
Procedure: DetermineS HIV-1/2 Test		_
3LSO_1F03N		W.,
information on form correct and complete?	Yes _	No
fine profilem and convertise without		
<u>wawes71810.0400 [400 lap 4.0000 90504.8</u>	TGSTR	
Information in register comes, end complete?	Yes	No.
Information agrees with results form?	Yes [1]	ho_
It no in either, problem and corrective action:		
Lot reamble or recorded/	Van 🗌	Nu
Quality Control per larmed?	Yee [No
find to either, corrective cetion:		
SHECKER STUDYOF TO HAWFRY STUDYOF	CLASC SATORS	
REGISTER Information in register correct and complete?	Yes 🗀	N:
Information segress with blood results register?	Yes -	N:
if no to either, prostern and current valuation;		3.5
	o care o tacco da	1112-010

Ongoing Monitoring

IIPTN LABORATORY QUALITY ASSURANCE MONITORING

D A T R	PTID (if applicable)	Event Code*	Description of Event**	Currective Action (Indicate if follow-up is required)	Initiut	Revier
					İ	
					+-	
					+	

"Specimen Management" SM, Results Reporting=RR, Technical Problems=TP, Complaints=C, Other=O

** Indicate Imposion of event: Mouseux Citiae, KCMC Citateal Lab, KCIdC Research Lab, Riolack Hailding, Transit